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THE SCIENCE OF CARE™

Symphony® Quick Start Guide

Scan to visit
Symphony Hub for
easy step-by-step
instructions and videos

INITIATE program

INITIATING LACTATION

AFTER BIRTH

**Pumping session
= 15 mins**

Stimulation phases,
expression phases and
pauses

1

Once **20 mL or more** is expressed
from **both breasts combined**,
during one pumping session

or

2

When physiological markers
of milk 'coming in'* become
apparent

*feeling of breast fullness

or

3

Day 6

**Switch to the
MAINTAIN program**
no later than day 6

When to switch from
INITIATE to **MAINTAIN**?

MAINTAIN program

BUILDING AND MAINTAINING LACTATION

**Pumping session
≥ 15 mins**

2-Phase Expression
(stimulation and
expression phase)

Note: This quick guide does not replace the complete instructions for use.
Please refer to the Symphony instructions for use for further information.

Wash hands thoroughly with soap and water before touching the breast pump, the pump sets and breasts.

INITIATE program

You can tell this program is running from these symbols on the display:

NOTE: Program ends automatically

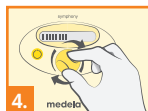
MAINTAIN program

You can tell this program is running from these symbols on the display:

Press the On/off button

wait

NOTE: Program does not end automatically

IMPORTANT!

For maximum effectiveness, efficiency and comfort, always double pump with your maximum comfort vacuum and correctly fitted breast shields. Go to www.medela.com/fittingguide to find your breast shield size.

Adjust to your **maximum comfort vacuum** by turning the vacuum knob to the right to increase suction until you experience slight discomfort. Next, turn it to the left to decrease suction until it is comfortable.

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© Medela AG 101046974/2024-04/B