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after
birth

8–12x
in 24h

Pumping session ≈ 15 mins
Stimulation phases,
pumping phases
and breaks

When to switch from INITIATE to MAINTAIN?

1

Once at least 20 ml or more is
expressed in three **consecutive**
pumping sessions

or

2

When physiological markers
of secretory activation*
become visible

*feeling of breast engorgement (milk comes in)

or

3

Day 6 +

**Switch to the
MAINTAIN program**
no later than day 6

MAINTAIN program

BUILDING AND
MAINTAINING
LACTATION

from
day 6

8–12x
in 24h


Pumping session ≥ 15 mins
2-Phase Expression
(stimulation and
pumping phase)

Note: This quick guide does not replace the complete instructions for use.
Please refer to the Symphony instructions for use for further information.

INITIATE program

You can tell this program is running from these symbols on the display:

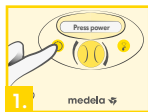Press power

Press the "Let-down" button to use the **INITIATE** program

Set vacuum


NOTE: Program ends automatically

MAINTAIN program

You can tell this program is running from these symbols on the display:

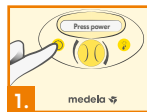Press power

Set a vacuum.
After 2 minutes, system switches automatically to the pumping phase (4.)


Re-set vacuum

NOTE: Program does not end automatically

After 10 seconds, the **MAINTAIN** program starts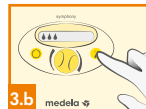

If milk begins flowing before the two minutes have passed, press the "Let-down" button to switch straight to pumping mode (4.).

IMPORTANT! SET THE VACUUM CORRECTLY AND CHECK/ADJUST IF NEEDED

Always use the maximum vacuum level comfortable. This helps produce optimal milk flow and makes pumping most effective.

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Medela AG